[ES] Información para el consumidor

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[EN] Consumer information

A full contact flexible wound dressing which provides an optimal wound healing environment and abnormal scar prevention.

Stratamed is a semi-occlusive, self-drying, non-sticky and transparent gel formulation that lightly bonds to the most superficial damaged skin.

When used as directed Stratamed dries to form a protective layer that is gas permeable and waterproof which hydrates and protects compromised skin areas and open wounds from chemical and microbial invasion.

Stratamed can be applied to fresh incisions and excisions, open wounds and compromised skin surfaces including those from post laser treatments, chemical peels, dermal abrasion, 1st and 2nd degree burns and radiation therapy.

Stratamed gel is bacteriostatic and inert.

For open wounds and compromised skin:

Stratamed dries to form a thin, flexible wound dressing which hydrates and protects affected areas. Stratamed helps create an optimal wound healing environment for superficial wounds and burns.

This environment leads to faster re-epithelialization and a reduced inflammatory response, which reduces recovery time post-procedure.

Stratamed does not adhere strongly to the wound bed or the newly formed granulating tissue.

Stratamed can be used with or without a secondary protective dressing.

For abnormal scar prevention and treatment:

Stratamed softens and flattens raised scars and allows scars to normalize the collagen synthesis process, producing a normal mature scar.

Stratamed is used to relieve itching and discomfort and to reduce redness and discoloration associated with damaged skin and scars.

Stratamed can be easily applied to all areas of the

skin, including exposed areas like the face and neck as well as joints and hairy areas without the need for shaving.

Stratamed was developed for the use on all types of superficial wounds, burns, skin tears and abrasions including those resulting from:

- General and cosmetic surgery
- Trauma
- Burns
- Radiation therapy
- Bites
- Grazes
- Laser therapy, including tattoo removal
- Chemical peels
- Dermal abrasion
- Cryotherapy
- Acne and other diseases of the skin

Intended use

Stratamed may be applied to open wounds, burns and areas where skin integrity has been compromised.

Stratamed is used for:

- The protection and hydration of superficial wounds, burns and compromised skin.
- The prevention and treatment of keloids, hypertrophic and excessive scars.

Stratamed may also be used in conjunction with invasive scar management options to improve overall results, including surgical excision and resurfacing, intralesional steroid injections, laser treatments and pressure garments.

Stratamed contains no steroids, alcohol, parabens or fragrances and is transparent and odourless.

Stratamed is suitable for children and people with sensitive skin.

Stratamed may be used on infants, during pregnancy and while breastfeeding.

Stratamed is intended for single patient use

Directions for use

On first use of 5g, 10g and 20g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50g tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the affected skin or wound is clean.

Gently pat dry as much excess exudate or wound fluid from the area as possible prior to gel application.

For best results Stratamed should be maintained in continuous contact with the skin (24 hours a day/7 days a week).

On damaged or broken skin not requiring a secondary dressing:

Apply a very thin layer of Stratamed to the affected area and allow the gel to dry. When applied correctly to the affected areas, Stratamed should be dry in 5–6 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

Stratamed should be applied once daily, or twice daily to exposed areas or as advised by your physician. Once dry, Stratamed may be covered by sunscreen, cosmetics, pressure garments or casts.

If Stratamed has been removed by washing, it should be reapplied.

On damaged skin or wounds requiring a secondary dressing:

Apply a thin layer of Stratamed then cover with the secondary dressing. Drying is not necessary.

Stratamed should be reapplied when changing the dressing or checking the wound progress, or as advised by your physician.

Recommended duration of treatment

Stratamed is recommended for immediate treatment post procedure and should be continued until the skin has fully recovered.

For resurfacing and ablative procedures, continued use is recommended for a minimum treatment of 30 days (24 hours a day/7 days a week) or until no further improvement is seen.

For abnormal scar prevention, Stratamed is recommended for a minimum treatment of 60–90 days (24 hours a day/7 days a week). Larger scars may take longer.

Continued use is recommended until no further improvement is seen.

How much Stratamed do I need?

Stratamed gel is an advanced formulation that requires substantially less product per application than typical creams or gels.

- **Stratamed 5g** is enough to treat a 2–4 cm linear incision twice per day for over 90 days.
- **Stratamed 10g** is enough to treat a 5–8 cm linear incision twice per day for over 90 days.
- **Stratamed 20g** is enough to treat a 10–12 cm linear incision twice per day for over 90 days or an area of 6×12 cm twice per day for over 60 days.
- **Stratamed 50g** is enough to treat an area of 12×15 cm, twice per day for over 60 days.

Caution

- For external use only.
- Stratamed should not be placed in contact with the eyes.
- Stratamed should not be applied over other skin treatments without the advice of your physician.

- Stratamed may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- Should your wound show signs of infection or fail to heal after 30 days of initial product application consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for highly exudative wounds, tunneling wounds or 3rd degree burns.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Do not use if the tube is damaged.
- Stratamed does not require special disposal methods. Please follow your local disposal regulations.
- Within the European Union, serious incidents related to the use of Stratamed should be reported to Stratapharma and the respective competent authority in the country the user is established.

STERILE UNTIL OPENED

Ingredients: Polysiloxanes, siloxane resin

 www.stratamed.com	
MD	Producto sanitario Dispositivo médico Medical device
	Consultar instrucciones de uso Consultar as instruções de uso Consult instructions for use
	Mantener fuera del alcance de los niños Mantenha fora do alcance das crianças Keep out of the reach of children
	Fecha de caducidad Date de validade Use by date
LOT	Código de lote Código do lote Batch code
	Fecha de fabricación Data de fabricação Date of manufacture
STERILE R	Esterilizado mediante irradiación Esterilizado por irradiação Sterilized using irradiation
	Sistema de barrera estéril individual Sistema de barreira estéril simples Single sterile barrier system
	No utilizar si el tubo está dañado Não use se o tubo está danificado Do not use if the tube is damaged
	Stratapharma AG, Aeschenvorstadt 57, CH-4051 Basel, Suiza / Suíça / Switzerland
EU REP	Freyr Regulatory GmbH Marie-Curie Straße 8, 79539 Lörrach, Alemania / Alemanha / Germany SRN: DE-AR-00047121
	Importer to the European Union: Stratapharma Limited, Clifton House, Fitzwilliam Street Lower, Dublin 2, D02 XT91, República de Irlanda / República da Irlanda / Republic of Ireland
Fecha de preparación de esta información / Data de preparação desta informação / Date of preparation of this information: 12/2025	

MS000E SPTPI05-1225